



Nozzles	Powders		Indications
AIR-N-GO® EASY SUPRA 120° Supplied as standard		"CLASSIC" Sodium bicarbonate-based Average particle size 76µm FLAVOURS: Neutral, Cola, Raspberry, Peppermint and Lemon	Finishing after scaling, Preparation for whitening, Cleaning of inter-proximal areas, fissures and crevices, Preparation of the tooth surface before etching, Cleaning of orthodontic brackets, Removal of supra-gingival biofilm, plaque, stains and temporary cement residues
		"PEARL" Calcium carbonate-based Average particle size 55µm FLAVOUR: Neutral	"Ultra-fast" cleaning of difficult stains and composites with no risk of damaging them Note: The absence of sodium makes it possible to treat sensitive patients or those following a salt-free diet
AIR-N-GO® PERIO EASY Supplied with the PERIO kit as an option		"PERIO" Glycine-based Average particle size 25µm FLAVOUR: Neutral	Removal of the sub-gingival biofilm Treatment of periodontal disease and peri-implantitis Pockets 3mm - 8mm
AIR-N-GO® PERIO Supplied as an option			Treatment of periodontal disease and peri-implantitis Pockets 8mm - 10mm
AIR-N-GO® PERIO MAINTENANCE Supplied as an option			Periodontal maintenance Pockets up to 4mm

For more information about good practice and the use of AIR-N-GO® easy, see our You Tube channel



COMMERCIAL OFFER

Standard

AIR-N-GO® easy: AIR-N-GO® easy handpiece with adaptor depending on the model (KaVo*, Sirona*, W&H*, Bien-Air*, Midwest*), 1x SUPRA 120°nozzle, 1x intro kit of 10 multiflavour sticks of "CLASSIC" powder and 2x sticks of "PEARL" powder, 1x AIR-N-GO® easy maintenance kit

*KaVo, Bien-Air, W&H, Sirona and Midwest are registered trademarks.

Optional

PERIO easy Kit: AIR-N-GO® PERIO easy nozzle and 1x bottle of 160g AIR-N-GO® "PERIO" glycine powder Supplied alone: AIR-N-GO® PERIO nozzle Supplied alone: AIR-N-GO® PERIO Maintenance nozzle









AIR POLISHING IS BECOMING EVEN SIMPLER WITH



Bacterial decontamination Reliability

A C t i V e Supra Freedom of Perional Perion of Movement

Signal Control of Perional A Signal Control of Movement

DUAL FUNCTIONS ACTEON® EQUIPMENT has created the new generation of portable air polishers, AIR-N-GO® easy with dual functions:

- "SUPRA function" for supra-gingival polishing: daily prophylactic polishing.
- "PERIO function" for sub-gingival polishing: maintenance, periodontal and peri-implant treatments.

This medical device allies simplicity with performance.

COMPLETE SOLUTION

4 nozzles for active control of the progress of periodontal disease.

Depending on your clinical needs, choose:

- Sodium bicarbonate-based "CLASSIC" powders
- Calcium carbonate-based "PEARL" powder
- Glycine-based "PERIO" powder



AIR N GO

Patient safety

SINGLE HANDPIECE Power and performance asy to convert

A single handpiece to cover a range of care and treatment

AIR-N-GO® easy combines essential functions with reliability to facilitate your everyday life:

- Fast start-up: direct connection to the high speed handpiece connector.
- Greater freedom of movement: 360° rotation, fast and accurate treatment of areas requiring care.
- Easier cleaning of the handpiece: main parts can be dismantled.

YOUR PARTNERS IN PREVENTING AND FIGHTING DISEASE

AIR-N-GO® nozzles, which result from the most advanced fluid dynamics modelling techniques, provide continuous, accurate and controlled spray whatever the clinical context. In daily practice these nozzles provide power and efficiency for the removal of supra- and sub-gingival biofilm.



- Tool suitable for any supra-gingival prophylactic treatment in combination with AIR-N-GO® "CLASSIC" or "PEARL" powders
- Guarantees a fast, efficient procedure for greater working comfort



Clinical case Dr P. ROUAS (France)



(optional PERIO easy kit)

- Innovative design in a "shoehorn" shape providing easy sub-gingival insertion
- A real asset for non-surgical, sub-gingival treatment of teeth and implants (3-8mm periodontal pockets)
- 5 seconds per face is all that is needed



Clinical case Dr C. LIM (U.K.)



- Reduced consumption of AIR-N-GO® "PERIO" fine powder particles in the maintenance phase
- Treatment of mucositis and return to a healthy clinical situation
- Action against the progression of periodontal disease





Clinical case Dr C. LIM (U.K.)



- Sub-gingival use for surgical and non-surgical treatments (tunnelling and open flaps)
- Powerful double lateral spray
- Decontamination of deep periodontal pockets (8-10mm)



Clinical case Dr A. PATEL (U.K.)

CUSTOMISED SOLUTIONS FOR YOUR CLINICAL NEEDS

AIR-N-GO® powders have been specially designed and studied to produce effective treatment that is gentle on all treated surfaces. All these powders are compatible with the AIR-N-GO® easy, no change of handpiece is required.

AIR-N-GO® "CLASSIC" POWDERS

Sodium bicarbonate-based

(≈ 76µm)

- Complete range of active supra-gingival powders which meet any patients' preferences
- Scrupulous cleaning giving instant results
- Action of removing coloured stains and bacterial plaque, even in areas that are difficult to access
- Better patient comfort: gentle and efficient with limited bleeding during treatment



5 fresh flavours based on 100% natural aromas or essential oils: Neutral, Cola, Raspberry, Peppermint and Lemon

COMPETITOR




Stratified particles with aggressive geometry (x1000)




Treated enamel shows numerous scratches (x300)

AIR N GO

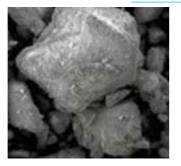

Particles with softened geometry (x1000)

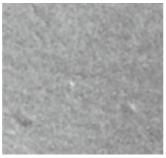

Treated enamel: surface preserved without impacts or scratches (x300)


AIR-N-GO®
"PEARL"
POWDER

Natural calcium carbonate-based (≈ 55µm)

- Supra-gingival polishing powder for delicate tissues protection
- Reduction in pH acidity caused by tissue inflammation; return to clinically healthy buccal flora
- Restoration of the natural shine and pearliness of dental enamel
- Fast cleaning of difficult stains and composites in total safety



Even the most sensitive patients with advanced gingivitis will benefit from "PEARL" microspheres.

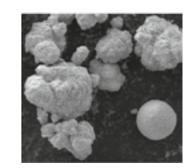

AIR-N-GO® "PEARL" (x1000)

Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos


AIR-N-GO®
"PERIO"
POWDER

Natural glycine-based (≈ 25µm)

- Exclusive use for sub-gingival treatments as a complement to the initial therapy
- Low abrasiveness preserving the treated surfaces (teeth and implants) and the most fragile anatomical areas
- Prevents the progression of the sub-gingival biofilm



- Plaque removal: during periodontal maintenance on exposed root surfaces without sacrificing substance
- Reduction of the bacterial load in the periodontal pockets

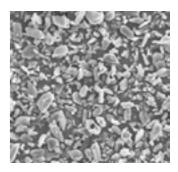

AIR-N-GO® "PERIO" (x1000)